


visuReal MASTER Update 1.5.1

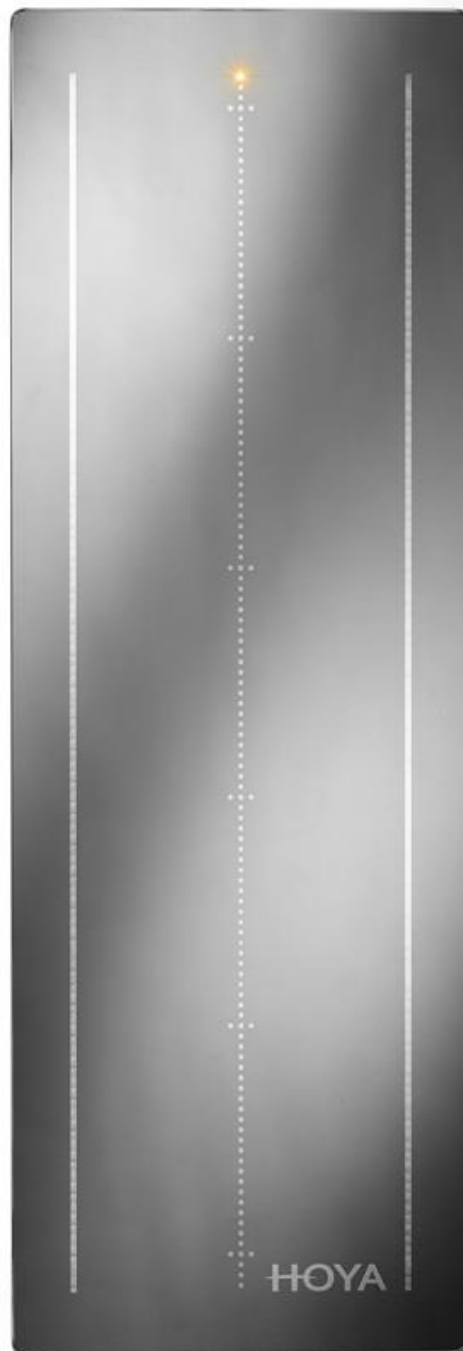

TOPICS


New Features:

1. New main menu item "Help"
2. Help page
3. Extended availability of the customer database from 48 to 72 hours
4. Desktop icon
5. Edit option for the reference (order ID) on the result page
6. Result e-mail: Subject header with reference / order ID
7. New update message (update now, skip, later)
8. FFFA sensibility reduction
9. Reduction of the measurement procedure pages

Information about New Features:

Point 1+2: New main menu item "Help" and link to Help page

Help around the clock! The new menu item Help provides you with additional information about the visuReal MASTER. It links to the user manual, FAQ section, video tutorial and an overview about new software features.

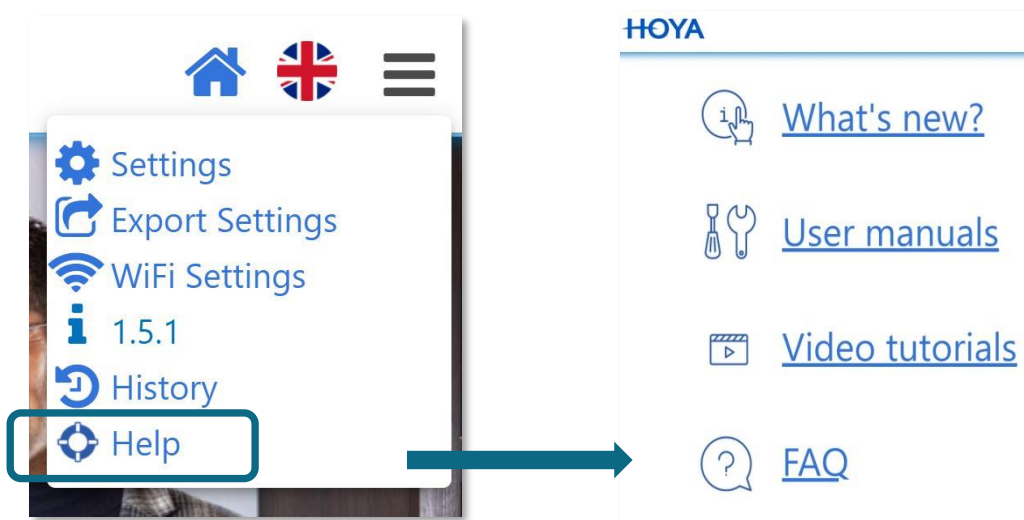

Point 3: Extended availability of the customer database from 48 to 72 hours

More time - more flexibility for you! As of version 1.5.1 the measurement data of your customers are available 72 hours before they will be deleted automatically (weekend excluded).



Point 4: Desktop icon

Small feature – great help! As of now, you can even easier and quicker access your visuReal Master software thanks to the new desktop icon. You will find this new icon on your desktop by adding the visuReal Master link to the desktop. How this is done depends on the browser you use.



Point 5: Edit option for the reference (order ID) on the result page

You made a mistake when entering the order ID at the beginning? No problem! You can edit and change the reference / order ID on the result page.




Point 6: Result e-mail: Subject header with reference/order ID

Easy allocation! When sending the results via e-mail, the subject header automatically contains the reference / order ID so that you can allocate the data easily.



Point 7: New update message

As of now, there is not only the red dot indicating an update. You will see a message on the visuReal Master start screen giving you three options: you can decide to update now, skip the update or do it later – here you can enter a date on which the message will pop up again to remind you to click on the red dot to perform the update.

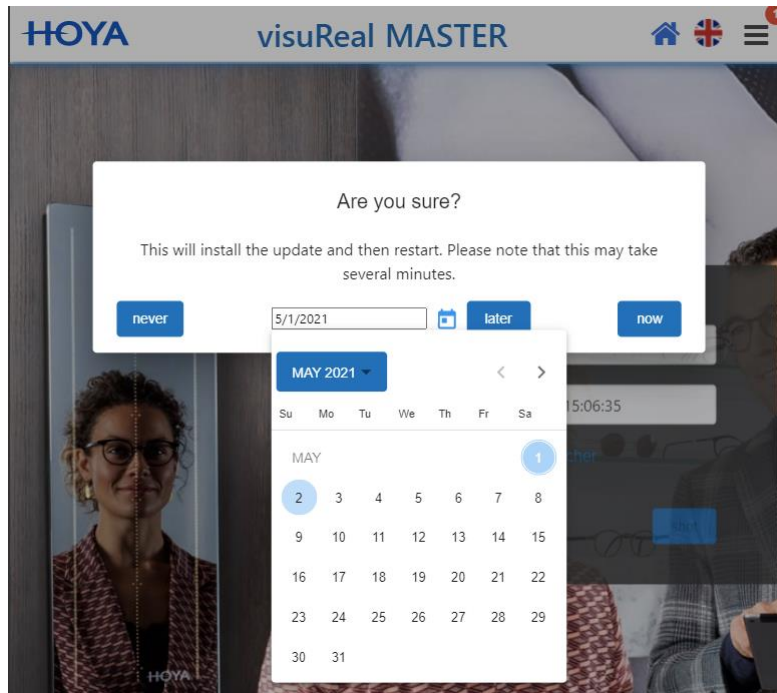


Point 8: FFFA sensibility reduction

The promised spreads are within the tolerance range.




Point 9: Reduction of the measurement procedure pages

The measurement pages are reduced from three to two pages. This allows a more user-friendly operation and gives a better orientation and overview.

Page 1: Check eye position cursors

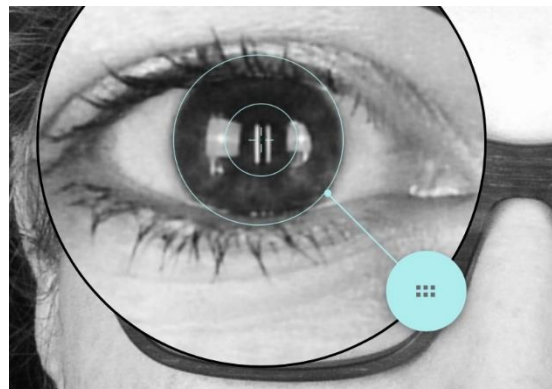

Page 2: Check the boxing measurement marks and the additional points – all on one page.

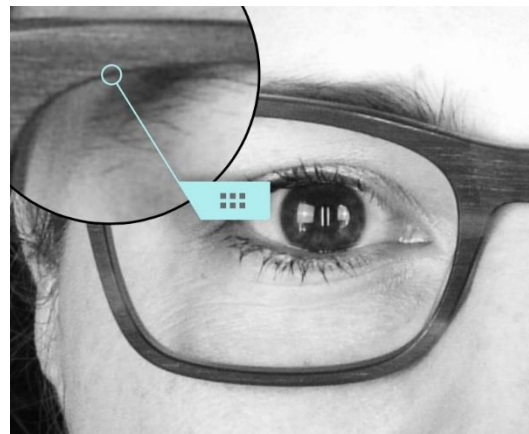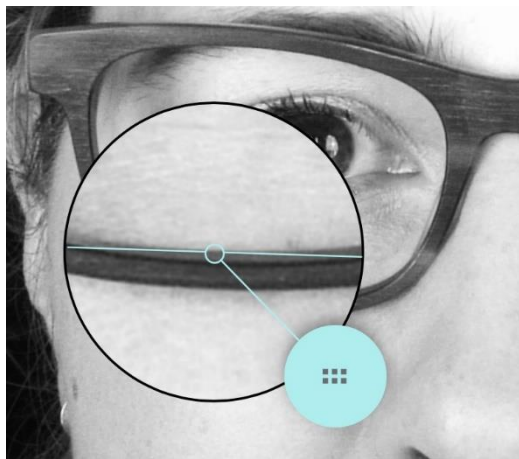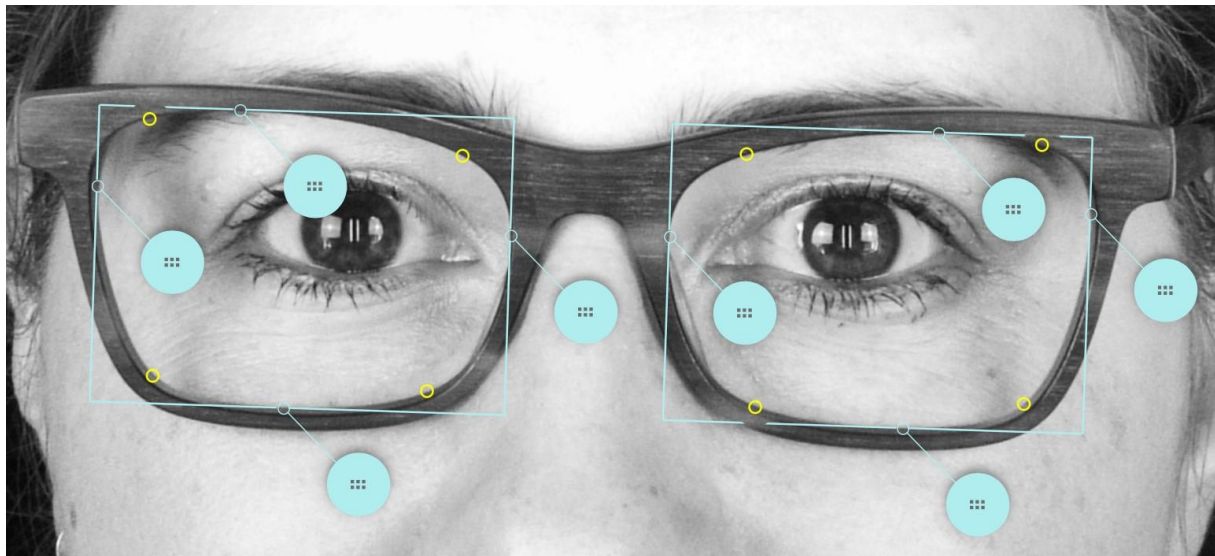

+49 (0) 39361 - 967 – 17



hotline@visuSolution.com



Lüderitzer Weg 6 39517 Tangerhütte OT Brunkau, Germany



www.visuSolution.com

Designed and developed by

